

## PRODUCT INSTRUCTION SHEET

**REFERENCE**     Tilton Shower Chair (142)  
                           142-2 / 142-4 (with fixed arms)  
                           142D-2 / 142D-4 (with detachable arms)

**DESCRIPTION**

The chair is constructed from nylon-coated all-steel welded frame, with a ‘H’ section base for added strength. Moulded polypropylene seat with drainage holes. Adjustable in height and portable with 2 or 4 braked castor wheels and fixed arms.

The Tilton is an aid for able bodied people to transport users/occupants quickly and safely between locations and to allow for easy manoeuvrability of the chair when unoccupied.

**TECHNICAL SPECIFICATIONS – 142-2 / 142-4**

Internal width between arms	19" (48cm)
External width for 142	21" (53cm)
Outside depth (front to back)	21" (53cm)
Seat height range:	19.5" (50cm) - 23.25" (59cm)
Backrest height:	14" (36cm)
Seat width x depth:	18" x 14" (46 x 36cm)
Armrest height from seat:	3.5" (9cm)
Castor diameter:	4" (10cm)
Product weight:	8.5kg (19lb)
Weight capacity:	165kg (26st)



*142-2 pictured above*



*142D-4 pictured left*

**TECHNICAL SPECIFICATIONS – 142D-2 / 142D-4**

Internal width between arms	19" (48cm)
External width for 142	21" (53cm)
Outside depth (front to back)	20" (51cm)
Floor to seat adj. height	20" x 25" (51 x 64cm)
Backrest height:	14" (36cm)
Armrest height from seat:	4.5" (11cm)
Castor diameter:	4" (10cm)
Product weight:	8.5kg (19lb)
Weight capacity:	165kg (26st)

## **PRODUCT INSTRUCTION SHEET**

### **SAFETY USAGE**

- Where height adjustment is required, ensure that there is no gap between the horseshoe clip and frame. The clip spindle must be fully located through to the outside of the frame.
- Where castors are fitted, ensure they are fully secure before, during and after use. Castor security must be checked periodically.
- Appliances to be used on a level surface only.
- Do not exceed the weight limit of 165kg (26 stone) or swing on the frames in any direction. The four castors must always be in contact with the floor.
- When issuing a shower chair, a risk assessment should be carried out to determine the suitability of the equipment for the user, with emphasis on the risk of entrapment.
- Ensure at least two castors are braked when the shower chair is stationary, particularly when a person is getting in and out of the chair.

### **CLEANING AND CARE**

- This product is intended for use in a wet room or shower.
- It is not intended for use outside and should not be immersed in water for long periods of time.
- To clean, use a mild detergent and warm water.
- Allow all equipment to dry thoroughly after use or cleaning.

### **WARRANTY**

Your Drive DeVilbiss Healthcare branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment.

This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist. This does not affect your statutory rights.

*Issued 2020/03*