

## PRODUCT INSTRUCTION SHEET

**REFERENCE**     Hallaton Mobile Shower Chair (141)

141-2 / 141-4 / 141D-2 / 141D-4 (19" / 48cm Seat Height)

141-4-457MM / 141D-4-457MM (18" / 46cm Seat Height)

141-4-406MM / 141D-4-406MM (16" / 41cm Seat Height)

### DESCRIPTION

The chair has a tubular metal frame with a moulded polypropylene seat with small drainage holes. The chair is available with fixed or detachable arms, and the shower chair is portable, with 2 or 4 foot-operated braked castors and a pushing handle.

The Hallaton is an aid for able bodied people to transport users/occupants quickly and safely between locations and to allow for easy manoeuvrability of the chair when unoccupied.



### TECHNICAL SPECIFICATION

- Seat width x depth:                             18" x 15" (46 x 38cm)
- Fixed seat height:                                16" / 18" / 19" (41 / 46 / 48cm)
- Outside depth x width                           20" x 21" (51 x 53cm)
- Internal width between arms:                19" (48cm)
- Maximum user weight:                        200kg (31.5st)
- Product weight:                                 8kg (18lb)

### SAFETY WARNING

- Ensure at least two castors are braked when the shower chair is stationary, particularly when a person is getting in and out of the chair.
- Where castors are fitted, ensure they are fully secure before, during and after use. Castor security must be checked periodically.
- Appliances to be used on a level surface only.
- When issuing a shower chair, a risk assessment should be carried out to determine the suitability of the equipment for the user, with emphasis on the risk of entrapment.
- Do not exceed the weight limit or swing on the frames in any direction. All four castors must always be in contact with the floor.

### CLEANING AND CARE

- This product is intended for use in a wet room or shower.
- It is not intended for use outside and should not be immersed in water for long periods of time.
- To clean, use a mild detergent and warm water.
- Ensure the product is dried thoroughly after use or cleaning.

### WARRANTY

Your Drive DeVilbiss Healthcare branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment.

This warranty is an expression of our confidence in the materials and workmanship of our products, and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist. This does not affect your statutory rights.